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INSTRUCTIONS FOR USE

SERVOX® placeholder acc. to Wollenberg SERVOX® tracheostomy placeholder

 Symbols used 19-20

EN

1. General Description

The SERVOX® placeholder acc. to Wollenberg and the SERVOX® tracheostomy placeholders are inserted in an artificially created tracheostomy and thus guarantee that the tracheostomy remains stable and open.

The SERVOX® placeholder acc. to Wollenberg has a 22 mm connector. The 22 mm connector gives you the option of adapting aids such as tracheostomy valves or HME filters, for example.

The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder are high-quality products made of medical-quality silicone and are supplied in non-sterile packaging. (The materials used meet national and relevant international standards.) They are latex and PVC-free and do not contain any metallic materials, so that there are no restrictions in the use of magnetic resonance imaging (MRI).

The material properties and the good workmanship mean that the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder are well tolerated tracheostomy products that can also be used on a long-term basis.

The silicone adapts itself exactly to the anatomical conditions in the trachea, thus guaranteeing a high level of comfort in wearing.

The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder are one-patient products.

2. Indication

The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder have been developed for the rehabilitation of patients after a tracheotomy or a laryngectomy. They are inserted into the tracheostomy and guarantee that the tracheostomy remains stable and open.

Note: The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder are only suitable for patients with surgically created chimney.

3. Contraindication

The use of the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder is not recommended in the event of known allergies to the material silicone.

If the tracheostomy is too tight or not suitably positioned, the use of alternative products is recommended.

Do not use this product if there are signs of mucosal or tissue damage to the tracheostomy.

4. Side-effects

The following side-effects may occur when the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder is used:

- irritation of tissue in the region of the tracheostomy through excessive pressure on the surrounding tissue

If you have any questions, or if complications should occur, please consult your doctor.

5. Application

Caution: Please do not use oily substances or silicone spray as lubricants, since these may change the material properties of the silicone.

Before inserting the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder, we recommend that they be cleaned in a suitable cleaning solution.

Before inserting the tube, ensure that you have selected the correct size.

Never use force when inserting or removing the product.

If the product is to be used during radiation or chemotherapy, the surrounding tissue must be carefully observed.

If you are using a voice prosthesis, make sure that the product does not touch the voice prosthesis and does not put any pressure on it. This could lead to damage to tissue.

When inserting and removing the product, make sure that it does not catch the voice prosthesis and possibly pull it out of the shunt.

When removing the product from the packaging, it is also important to ensure that the packaging is unopened and intact. Before inserting the product, check it for possible damage.

a. Inserting and removing the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder

Insertion:

To insert the placeholder, it is recommended that the upper part of the half-cannula should be folded inwards. When the half-cannula has been folded, it must be held in that position, as it would otherwise open up again to the original position. When they are in this state, the placeholders are to be inserted in the tracheostomy. To do this, the lower part of the half-cannula is first inserted into the tracheostomy as far as the transition of the half-cannula into the cannula tube. The placeholder is then slowly released. The upper part of the half-cannula folds open and nestles against the wall of the trachea.

The SERVOX® placeholder acc. to Wollenberg can then be secured with a tube carrying strap. The carrying strap must be positioned in such a way that the freedom of movement of the head is not restricted and that the SERVOX® placeholder acc. to Wollenberg is securely held in the tracheostomy.

Please use only recommended tube carrying straps. Tube carrying straps with sharp-edge holding devices may damage the retaining eyelets.

Removal:

The placeholders are removed by pulling them carefully out of the tracheostomy.

b. Cleaning and care

Notes:

- The products must not be sterilised, and must not be exposed to direct sunlight. This may lead to the destruction of the material.
- Before cleaning, any aids mounted on the standardised 22 mm connector must be removed and cleaned or disposed of separately. Please observe the recommendations of the manufacturer.
- To prolong the service life of the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder, they should be cleaned at least 1-2 x per week, if necessary more frequently. Before cleaning, the product must be removed from the tracheostomy.

Warning: The cleaning of the SERVOX® placeholder must be done out of the tracheostomy, to avoid aspiration!

The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder can be heated to temperatures of up to 65 °C in a water bath. The SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder can be cleaned using mild household cleaning agents. We recommend the use of a cleaning solution for tracheostomy tubes. Rinse the products thoroughly under running water after they have been cleaned and dry them with a lint-free cloth.

Caution: Under no circumstances may cleaning be carried out with aggressive household detergents or high-percentage alcohol, since these may alter the

material properties. Furthermore, do not use any disinfectants that release chlorine, or agents that contain strong lyes.

Secretions sticking to the tube can be carefully removed using a cleaning brush. Make sure you do not apply excessive pressure. To prevent damage to the silicone, no mechanical or sharp-edged aids may be used, for example scrapers, knives, fingernails or the like.

After cleaning, the aid must be stored in a dry place.

Even after they have been cleaned several times, in accordance with fixed standards and taking account of application notes, the SERVOX® placeholder acc. to Wollenberg and SERVOX® tracheostomy placeholder can be used several times with one patient for a period of at least 29 days and a maximum of 6 months.

6. **Warnings:**

- Do not carry out any alterations to your product!
- If the product should have any damage such as sharp edges, cracks or the like, you are advised not to use it.

Disregarding these notes can have severe consequences for health, including potentially fatal injuries. If the product should have been manipulated, the manufacturer can accept no liability in the event of injury.

7. **Safety**

Each product is assigned a batch number, which is marked on the outer packaging and on the label, to allow unequivocal traceability of the product. The batch number must be entered, or the enclosed label affixed, in the patient ID card supplied. It is recommended that the patient should always carry his ID card with him.

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