# SERVOX® Tracheo Tape Neo plus

DE	Zweiteiliges gepolstertes Kanülentrageband mit Klettverschluss (weiß)
EN	Two-piece padded tube holder with Velcro (white)
FR	Bande rembourrée portecanule en deux parties, à fermeture auto-agrippante (blanche)
IT	Nastro portacannula imbottito in due pezzi con velcro (bianco)
ES	Cinta de fijación para cánula, acolchada, de dos piezas con cierre de velcro (blanca)
NL	Tweedelige gevulde canule draagband met klittenbandsluiting (wit)
EL	Φέρουσα κορδέλα σωλήνα δύο τμημάτων, με μαλακή επένδυση και Velcro (λευκή)
BG	Поддържаща лента за канюли, от две части, с подплата, с велкро закопчалка (бела)
HR	Dvodijelna pojačana fiksacijska traka za kanile s čičkom (bijela)
PL	Dwuczęściowa opaska do kanułu (tracheostomijnych) z miękkim posyciem, zapinana na rzepy (kolor biały)
PT	Fita de suporte da cânula almofadada com fecho de velcro (branco)
SL	Dvodelni oblazinjeni nosilni trak za kanilo s sprijemalnim zapiralom (bele barve)
HU	Kétféreszes párnázott kanültartó pánt tépőzárral (fehér)

**REF XXXXX**

**LOT XXXXXX**

**PZN xxxxxxxx**

**JJ-MM**

**UDI**

(xx)xxxxxxxxxxxxxx  
(xx)xxxxxx  
(xx)xxxxxx

**Orbisana Healthcare GmbH**  
Biberweg 24-26 | 53842 Troisdorf | Germany  
Tel. +49 (0)2241 9322-0  
info@orbisana.de | www.orbisana.de

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**EN Intended use | indication | description**

The SERVUX® Tracheo Tape Neo plus is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open. The SERVUX® Tracheo Tape Neo plus secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Reverse side of polyester tube holder with cotton coating
- Suitable for use on neonates and children
- single-use, do not reuse (risk of infection)
- The maximum duration of use is 48 hours, then dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck
- Soft padding to prevent decubitus

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### EN Application

- Open all Velcro fastenings.
- Slide the shorter end of the band through one of the eyelets on the neck flange from below and close the Velcro.
- Slide the longer end of the band through the other eyelet on the neck flange from below and close the Velcro.
- Place the longer part of the band around the neck.
- Connect both parts of the band with the Velcro.
- Use the Velcro to adjust the length of the band.
- Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- Cut off any excess band.
- To remove, release the Velcro and take off the band.

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### EN Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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