SERVOX® Tracheo Tape M duo

DE	Zweiteiliges gepolstertes Kanülentrageband mit Klettverschluss (weiß)
EN	Two-piece padded tube holder with Velcro (white)
FR	Bande rembourrée portecanule en deux parties, à fermeture auto-agrippante (blanche)
IT	Nastro portacannula imbottito in due pezzi con velcro (bianco)
ES	Cinta de fijación para cánula, acolchada, de dos piezas con cierre de velcro (blanca)
NL	Tweedelige gevulde canuldraagband met klittenbandsluiting (wit)
EL	Φέρουσα κορδέλα σωλήνα δύο τμημάτων, με μαλακή επένδυση και Velcro (λευκή)
BG	Поддръжка лента за канюли, от две части, с подплата, с велкро закопчалка (бяла)
HR	Dvodieljna pojasna traka za kanile s čičkom (bijela)
PL	Dwuczęściowa opaska do kaniumi tracheostomijnych z miękkim poszyciem, zapinana na rzepy (kolor biały)
PT	Fita de suporte da cânula almofadada com fecho de velcro (branco)
SL	Dvodielni obložljivi nosilni trak za kanilo s sprijemalnimi zapiralom (bele barve)
HU	Kétrészes párnázott kanültartó pánt tépőzárral (féhér)

REF XXXXX

PZN xxxxxxxx

UDI

(xx)xxxxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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EN Intended purpose | indication | description
The SERVOX® Tracheo Tape M duo is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open. The SERVOX® Tracheo Tape M duo secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Reverse side of polyester tube holder with cotton coating
- Suitable for all age groups depending on the neck circumference
- Single-use, do not reuse (risk of infection)
- The maximum duration of use is 48 hours, then dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck
- Soft padding to prevent decubitus

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EN Application

- Open all Velcro fastenings.
- Slide the shorter end of the band through one of the eyelets on the neck flange from below and close the Velcro.
- Slide the longer end of the band through the other eyelet on the neck flange from below and close the Velcro.
- Place the longer part of the band around the neck.
- Connect both parts of the band with the Velcro.
- Use the Velcro to adjust the length of the band.
- Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- Cut off any excess band.
- To remove, release the Velcro and take off the band.

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EN Please note

- Regularly check the fit and adjust where necessary.
- Re-measure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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