SERVOX® Tracheo Tape soft plus

- DE Gepolstertes Kanülentrageband mit Haken (weiß)**
- EN Padded tube holder with hooks (white)**
- FR Bande rembourrée porte-cannule avec crochet (blanche)**
- IT Nastro portacannula imbottito con gancho (bianco)**
- ES Cinta de fijación para cánula, acolchada, con gancho (blanca)**
- NL Gevulde canuledraagband met haakjes (wit)**
- EL Φέρουσα κορδέλα σωλήνα με μαλακή επένδυση και γάντζο (λευκή)**
- BG Поддържаща лента за канюли, с подплата, с кукички (бяла)**
- HR Pojaštvena fiksacijska traka za kanile s kukicama (bijela)**
- PL Opaska do kanili tracheostomijnych z miękkim posyciem i z haczykiem (kolor biały)**
- PT Fita de suporte de cânula almofadada com gancho (branco)**
- SL Oblazinjeni nosilni trak za kanilo s kavljí (bele barve)**
- HU Párnázott kanültartó pánt kapoccsal (fehér)**

xx cm

REF XXXXX LOT XXXXXX

PZN xxxxxxxx yyyy-mm

(xx)xxxxxxxxxxxxxx
(xx)xxxxxxx
(xx)xxxxxx

Orbisana Healthcare GmbH
Biberweg 24-26 | 53842 Troisdorf | Germany
Tel. +49 (0)2241 9322-0
info@orbisana.de | www.orbisana.de

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EN Intended use | Indication | description
The SERVOX® Tracheo Tape soft plus is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open. The SERVOX® Tracheo Tape soft plus secures the aid in the tracheostoma and prevents it from accidentally slipping out.
• Reverse side of tube holder with cotton coating
• Suitable for all age groups depending on the neck circumference
• single-use, do not reuse (risk of infection)
• The maximum duration of use is 48 hours, then dispose of in household waste
• Can be seamlessly adjusted to the circumference of the neck
• Soft padding to prevent decubitus

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EN Application

- 1 Place the band around the back of the neck
- 2 Insert both hooks into the eyelets on the neck flange from below. The back of each hook is facing the throat.
- 3 Use the slider to adjust the length of the band.
- 4 Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- 5 To remove, unhook one side of the band from the neck flange.

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EN Please note

- Regularly check the fit and adjust where necessary.
- Re-measure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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