SERVOX®

Tracheo Tape M soft

DE Gepolstertes Kanülentrageband mit Klettverschluss (weiß)
EN Padded tube holder with Velcro (white)
FR Bande rembourrée porte-canule à fermeture auto-agrippante (blanche)

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REF XXXXX

PZN XXXXXXXX

(xx)xxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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EN Intended use | indication | description

The SERVOX® Tracheo Tape M soft is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open. The SERVOX® Tracheo Tape M soft secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Polyester tube holder

• Suitable for all age groups depending on the neck circumference

• Single-use, do not reuse (risk of infection)

• The maximum duration of use is 48 hours, then dispose of in household waste

• Can be seamlessly adjusted to the circumference of the neck

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EN Application

- Open all Velcro fastenings.
- Place the band around the back of the neck.
- Slide both ends of the band through the eyelets on the neck flange from below and close the Velcro. The Velcro must be on the outside.
- Use the Velcro fasteners to adjust the length of the band.
- Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- To remove, release the Velcro on one side and pull the band out of the eyelet.

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EN Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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