Tracheo Tape M Neo

- DE** Zweiteiliges gepolstertes Kanülentrageband mit Klettverschluss (weiß)
- EN** Two-piece padded tube holder with Velcro (white)
- FR** Bande rembourrée portecanule en deux parties, à fermeture auto-agrippante (blanche)
- IT** Nastro portacannula imbottito in due pezzi con velcro (bianco)
- ES** Cinta de fijación para cánula, acolchada, de dos piezas con cierre de velcro (blanca)
- NL** Tweedelidig gevulde canuledraagband met klittenbandsluiting (wit)
- EL** Φέρουσα κορδέλα σωλήνα δύο τμημάτων, με μαλακή επένδυση και Velcro (Λευκή)
- BG** Поддържаща лента за канюли, от две части, с подплата, с велкро закопчалка (бяла)
- HR** Dvodijelna pojačučena fiksacijska traka za kanile s čičkom (bijela)
- PL** Dwuczęściowa opaska do kaniuli tracheostomijnych z miękkim posyciem, zapinana na rzepy (kolor biały)
- PT** Fita de suporte da cânula almofadada com fecho de velcro (branco)
- SL** Dvodielni obložajeni nosilni trak za kanilo s prijemnim zapiralom (bele barve)
- HU** Kétrészes párnázott kanültartó pánt tépőzárral (fehér)

REF XXXXX

PZN xxxxxxxx

UDI

(xx)xxxxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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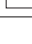
EN Intended use | indication | description
The SERVOX® Tracheo Tape M Neo is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open.
The SERVOX® Tracheo Tape M Neo secures the aid in the tracheostoma and prevents it from accidentally slipping out.
• Reverse side of polyester tube holder
• Suitable for all age groups depending on the neck circumference
• Single-use, do not reuse (risk of infection)
• The maximum duration of use is 48 hours, then dispose of in household waste
• Can be seamlessly adjusted to the circumference of the neck
• Soft padding to prevent decubitus

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EN Application

- Open all Velcro fastenings.
- Slide the shorter end of the band through one of the eyelets on the neck flange from below and close the Velcro.
- Slide the longer end of the band through the other eyelet on the neck flange from below and close the Velcro.
- Place the longer part of the band around the neck.
- Connect both parts of the band with the Velcro.
- Use the Velcro to adjust the length of the band.
- Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- Cut off any excess band.
- To remove, release the Velcro and take off the band.

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EN Please note

- Regularly check the fit and adjust where necessary.
- Remeasure the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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